

# HemoBand®

- Disposable
- Single Use Only
- Latex Free
- Sterile

The HemoBand is an adjustable, single-use band with a molded pressure pad. It is used for control of bleeding following needle removal. The band is designed for ease of use and patient comfort. The HemoBand also frees the dialysis staff from prolonged contact due to holding of needle sites. The band is wide to be comfortable to the patient while applying ample pressure for clotting of needle sites. The translucent plastic provides easy visualization of the needle site for ease in placement.

**DFU:** The HemoBand is used to provide pressure hemostasis of arterial, venous and dialysis access needle puncture sites.

**WARNING:** After placing HemoBand, check the graft for flow. A bruit must be detected to ensure graft patency. If a bruit is not heard the HemoBand should be loosened until the bruit returns.

Never place an object other than a single sterile gauze pad under the HemoBand pressure pad.

**CAUTION:** Federal law restricts this device to sale by or on the order of a physician.

**REF:** Sterile: HB-1M  
**Package Contents:** 12 HemoBands  
1 Instructions for Use



MANUFACTURER



CAUTION

## HemoBand Incorporated

515 N.W. Saltzman Rd. #797  
Portland, OR 97229-6098 U.S.A.  
1-800-297-8485 • FAX: 1-503-617-1823  
[www.hemoband.net](http://www.hemoband.net)

World wide patent pending  
PATENT #: US 5,269,803

REV1901

# HemoBand® Sterile HB-1M

## Standard Length - 14"

### Sterile Hemostasis Pressure Band



**2** DO NOT REUSE

Always use a new single use HemoBand for each procedure.  
Dispose of after each use.

Discover our complete line of quality products  
and re-order at [www.hemoband.net](http://www.hemoband.net).



HemoBand - Non-Sterile  
HB-NSXL  
(Extra Long Length - 17")




HemoBand - Non-Sterile  
HB-NS  
(Standard Length - 14")

# Instructions for Use


# HemoBand®




**1** **PREP:** Place a HemoBand **loosely** around the arm near the needle puncture site(s). The end of the strap is inserted into the adjustable locking latch of the HemoBand. Adjust by gently pulling up on the strap.

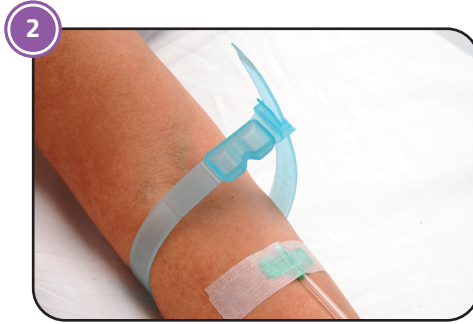

**2** **NOTE:** If the second HemoBand is to be used at the same time, place it loosely around the arm and follow the same procedures for USAGE. The second band should be placed under the tubing of the other needle so as not to interfere with its removal.

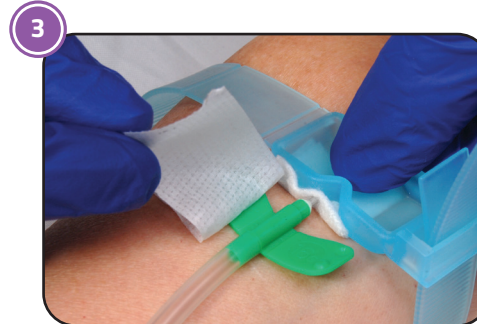

**3** Prepare to remove the needle(s). Place a sterile gauze dressing over the needle puncture site. Position the HemoBand pressure pad so that the V-shaped groove is directly over sterile gauze dressing covering the needle puncture site. The HemoBand is **gently** tightened while maintaining the pressure pad's position over the gauze dressing and needle puncture site.

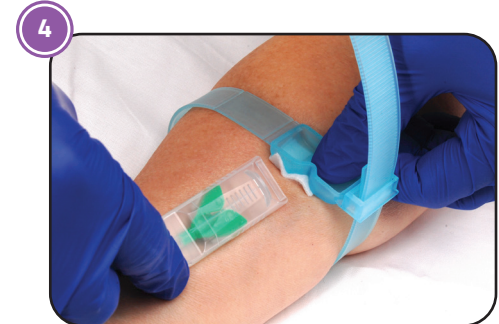


**4** To remove the needle, place your thumb into the recessed back of the pressure pad, **but do not apply pressure with your thumb at this time**. Now, use your other hand to remove the needle.




**5** Any bleeding that may occur can be easily controlled with additional pressure from your thumb on the pressure pad, after the needle has been removed.



**6** Make adjustments to the band pressure as needed. Bleeding should now be controlled. Any fine adjustment of the HemoBand can now be made for patient comfort or persistent bleeding.

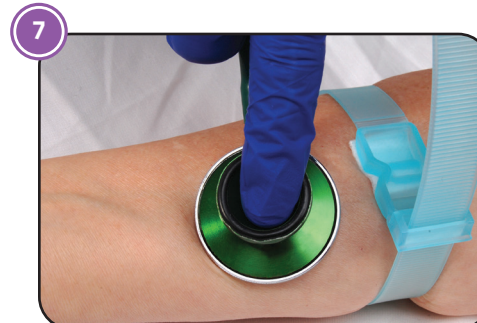

**7** Check the graft for flow. This must be done to ensure that the graft has not been totally occluded. Using a stethoscope listen over the graft for a bruit. A bruit is the sound of blood rushing through the graft. It will sound like a whooshing noise with each heart beat. If a bruit is not heard the HemoBand should be loosened until the bruit returns.

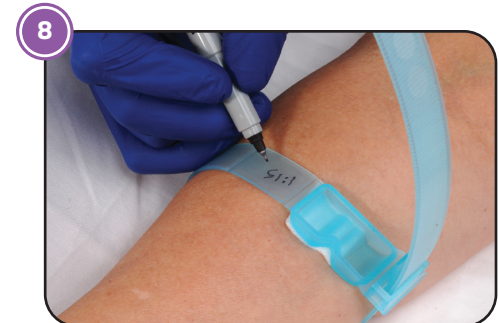

**8** As an option, write the estimated time to remove the HemoBand directly on the HemoBand surface using a permanent marker. Most needle site bleeding will be controlled after five to ten minutes. It is recommended that you follow established holding times at your facility.



**9** After hemostasis is achieved, the HemoBand may be removed. The band is removed by slowing loosening the latch. **Discard into an appropriate biohazard container.**

**2** DO NOT REUSE

Always use a new single use HemoBand for each procedure. Dispose of after each use.